



Oximeter Upload Instructions Wrist Ox

The first 5 steps can be done anytime prior to uploading a test

1. Go to <https://protecs.scriptitonline.com>
2. Enter ID and password
3. Click “Start a new patient filing”
4. Follow the instructions to insert patient information
5. We do not require a SSN. Just insert nine random digits.

Steps 6-22 are to be performed when uploading

6. Attach 9-pin connector into computer
7. Connect oximeter to opposite end of cable.
8. Open “Protecs Universal” software on your desktop
9. If you get the following “Publisher not verified. Are you sure you want to run?”
Uncheck the box on the left (always show this warning) and click “run”
10. Enter ID and password
11. Select your patient
12. Pick the correct COM Port
13. Pick Device Type “Wrist Ox”
14. Check pertinent patient data (overnight, oxygen, room air or other)
15. Click on “Get Data” box
16. Numbers will start counting in lower right
17. Display box will appear “Download Complete Processing Sessions”. Click “OK”
18. Window will display sessions available
19. Click to highlight which sessions are to be included in report
20. Click “upload” once
21. Message will appear when data has been transmitted successfully
22. Fax AOB and physician order to 1-866-925-1962 if not previously done.