



Pulse Ox download Instructions

1. Connect 9-pin cable into computer.
2. Connect pulse oximeter to 9-pin cable.
3. Open CCare application.
4. Fill in required information.
5. Click on Get Data box – button on upper right hand corner.
6. Please replay memory will appear.
7. Place pulse oximeter in communication mode: push the little blue button then big blue button (power button) and hold for several seconds and then release
8. Numbers will start counting.
9. Window will display sessions in machine.
10. Click to highlight which sessions to be included.
11. Click on mode of testing: on Room Air, on 2 lpm etc.
12. Choose which type of report: standard, 2 hour, 4 hour
13. Enter username and password.
14. Submit data – please once click only.
15. Message will appear when data has been transmitted successfully.