



SETTING THE DATE & TIME 920M SERIES OXIMETERS

DATE & TIME

1. Press and hold the small button while turning on the oximeter the release both buttons
2. Oximeter will read "Clr No"
3. Press large button
4. Display reads "y"
5. Press small button until you have the correct year
6. Press large button
7. Display reads "nn"
8. Press small button until you have the correct month
9. Press large button
10. Display reads "dd"
11. Press small button until you have the correct day
12. Press large button
13. Display reads "hr"
14. Press small button until you have the correct hour
15. Press large button
16. Display reads "nn"
17. Press small button until you have the correct minute
18. Press large button
19. Display reads "Prn 00"
20. Press large button
21. Display reads "- -"
22. Press and hold large button until oximeter powers off

INDEPENDENT DIAGNOSTIC TESTING FACILITY

TEL: 888-662-7157 • testing@proteclabs.com